

sleepcube

AUTOADJUST



...feel refreshed!

sleepcube

AUTOADJUST

The new Sleepcube AutoAdjust from DeVilbiss has all the functions you would expect from a professional auto adjusting device yet manages to fit them into the new sleek Sleepcube design. With its unique auto adjust algorithm this device is equally suitable for performing overnight titrations in the sleep laboratory or at home giving the patient maximum comfort as it adjusts its pressure in response to apnoeas, hypnoeas and snoring finding the optimum pressure settings throughout the night. As part of the Sleepcube family the AutoAdjust has all the comprehensive features already found on the standard model. It uses the same intuitive control panel for a fast and simple set-up. The optional DV5HH heated humidifier can be easily connected and operated from the AutoAdjust.

Connecting the SmartLink compliance module allows overnight data to be recorded and downloaded to the DeVilbiss LT Software either by direct connection or by smart card. The SmartLink also has an interface for oximetry to be added and this data too can be sent to the LT Software for analysis and reporting.



The simple LED push button display, makes the Sleepcube easy to use, even in the dark



Back of the Sleepcube – showing filter, patient tubing outlet and download outlet



Sleepcube with integrated heated humidifier



Sleepcube AutoAdjust with SmartLink unit attached

SPECIFICATIONS

Dimensions (H x W x L)	10.8 x 16.5 x 17.8 cm	Storage & Transportation Temperature Range	-40° C to +70° C
Weight	1.18 kg	Humidity Range	0% to 95% R.H. non-condensing
Auto Adjust CPAP Pressure Range	3-20 cm H ₂ O	Sound Level	26 dB(A)
Operating Temperature Range	5° C to 40° C	Maximum Power Consumption	65 W from AC power source (PAP device only)
Humidity Range	0% to 95% R.H. non-condensing	Electrical Requirements	100-240 VAC, 50/60 Hz
Atmospheric Conditions	Sea level to 9,000 feet		

Distributor:



CE 0044

Sunrise Medical Limited
High Street Wollaston/Stourbridge
West Midlands DY8 4PS
England

Phone + 44 (0) 13 84 44 68 58
Fax + 44 (0) 13 84 44 66 28
www.devilbisshealthcare.com
enquiries@devilbisshc.com